



# Ultra Lightweight Installation Guide

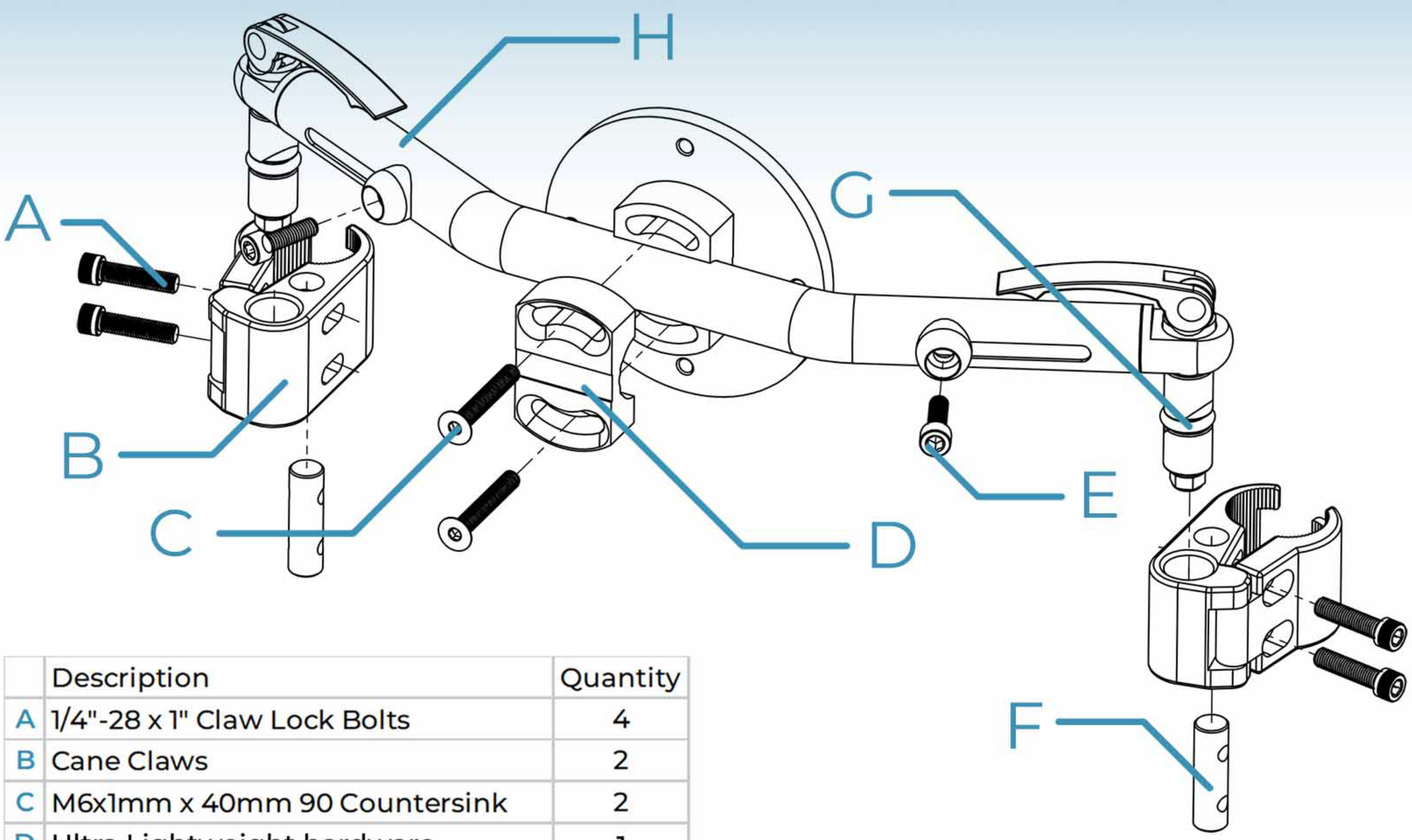


	Description	Quantity
A	1/4"-28 x 1" Claw Lock Bolts	4
B	Cane Claws	2
C	M6x1mm x 40mm 90 Countersink	2
D	Ultra Lightweight hardware	1
E	M6x1mm x 20mm Adjustment Bolts	2
F	Cane Claw Shaft	2
G	Rubber O-Ring 1/8" x 7/16" x 11/16"	2
H	Crossbar	1


Light Weight as received:

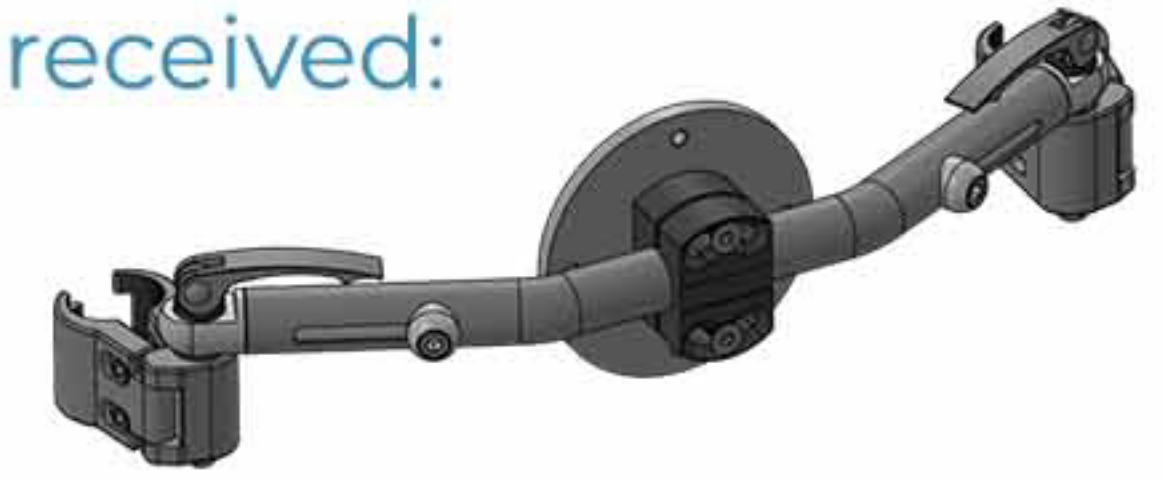

## 1 Adjust the width of the Crossbar.

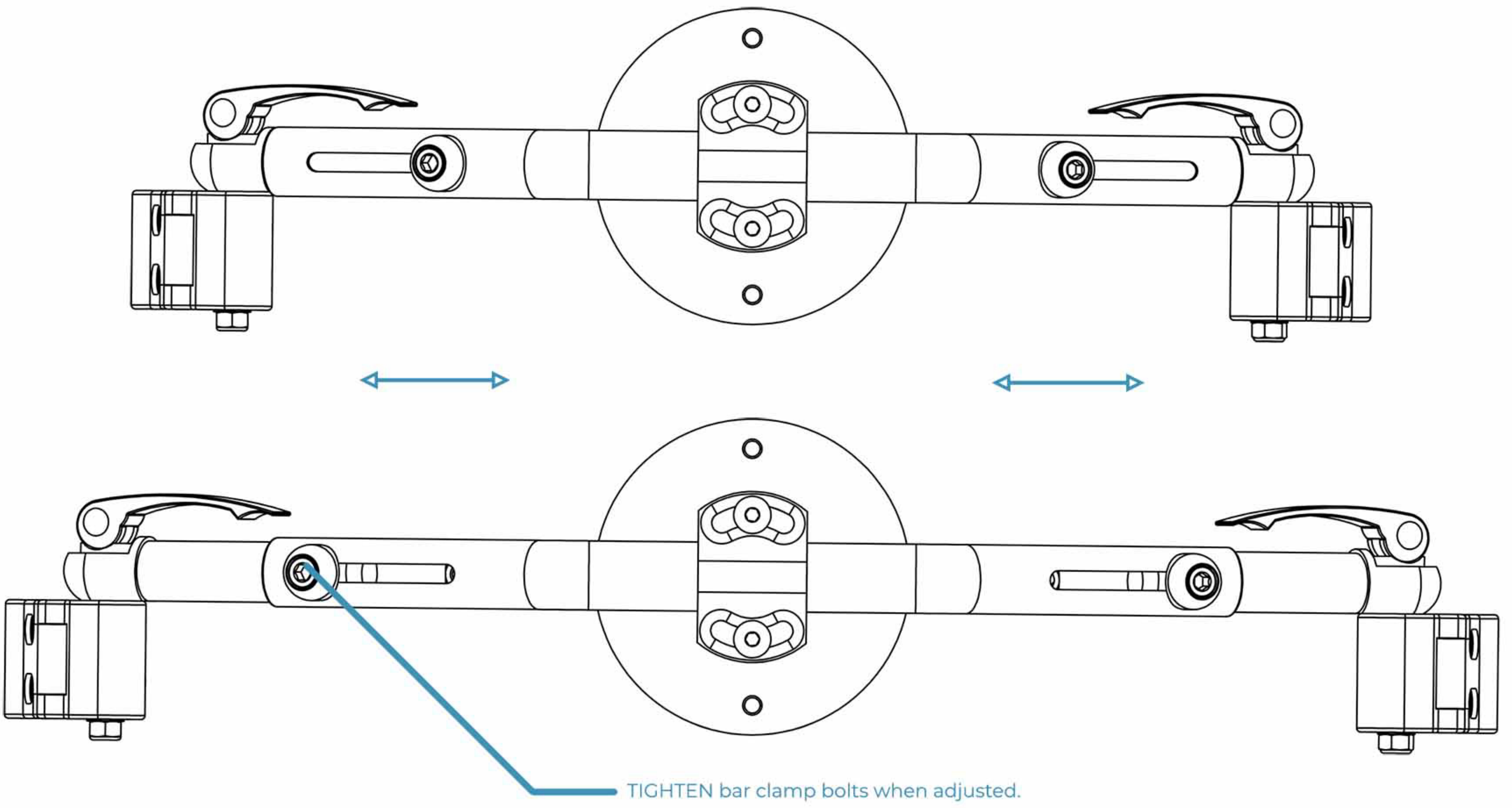

## 2 Secure cane claws to the wheelchair canes.

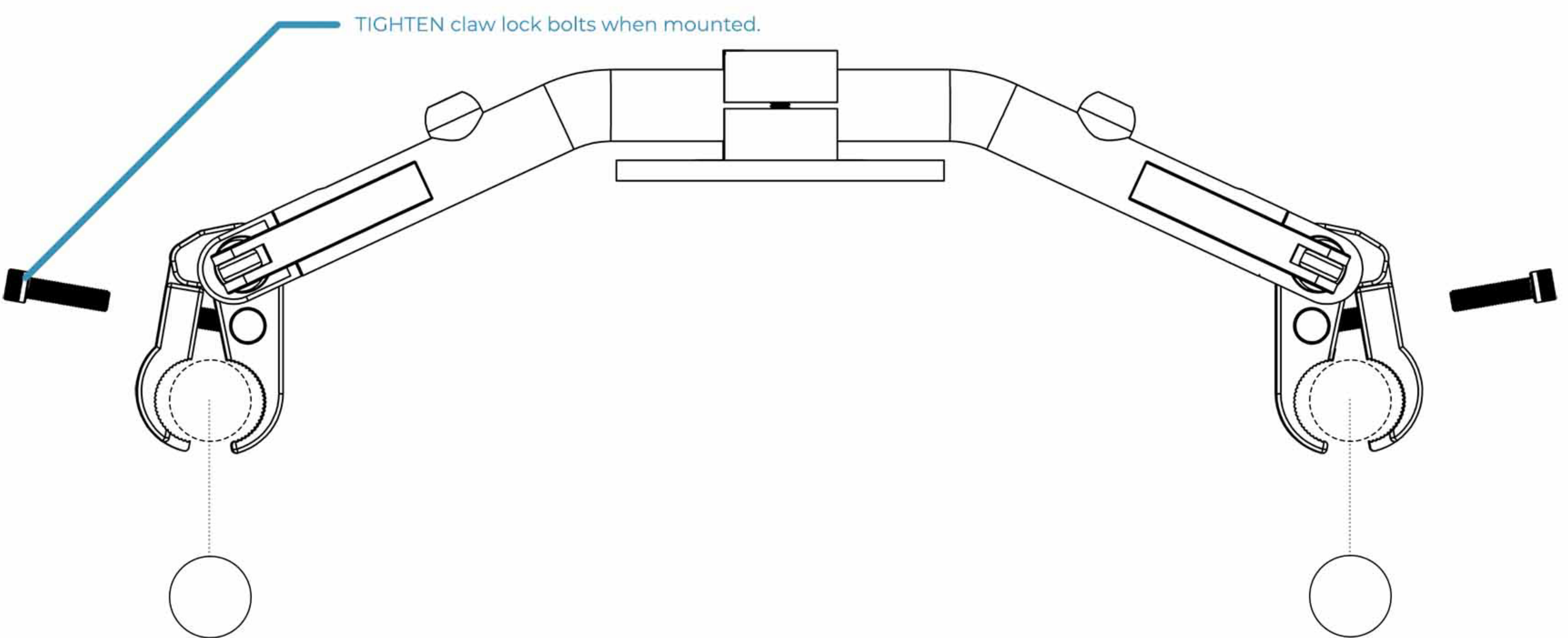

## 3 Adjust seat rotation.

