



Get the app!  
**Matrix Keen3D**

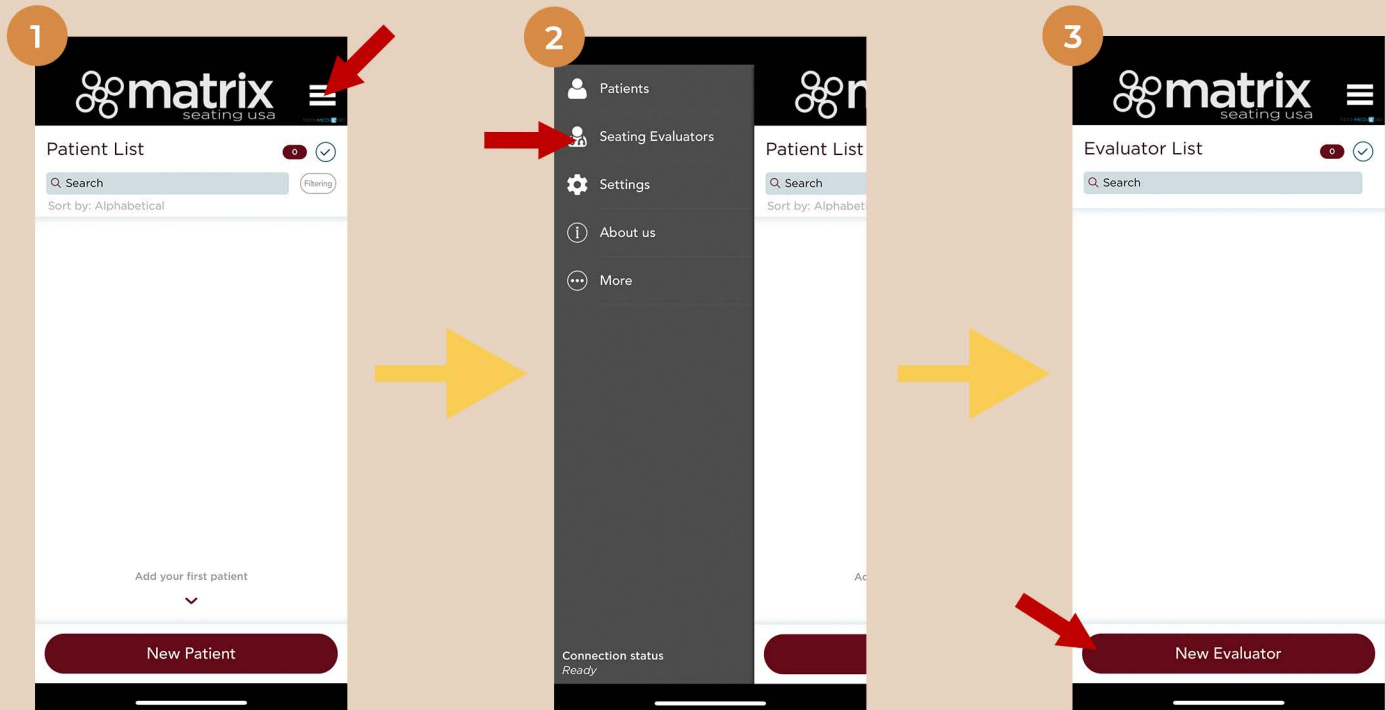


Upon opening the app you will be greeted with your **Patient List**. Before you can add new patients you must select an evaluator.

Do this by pressing the **Menu** button in the top right. This will reveal the **Seating Evaluators** tab.


Press the **Seating Evaluators** tab and select your name, or press the **New Evaluator** button to add it.

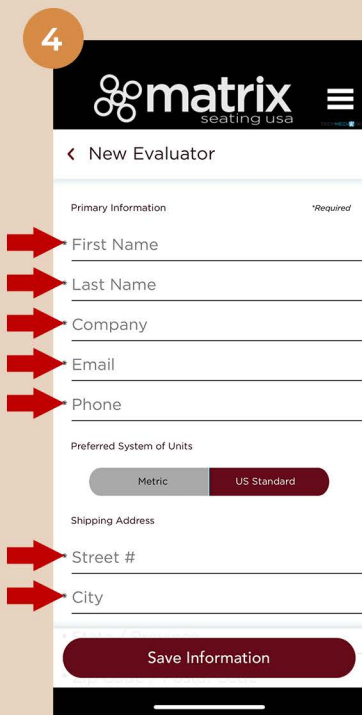


If adding a **New Evaluator**, fill out the required information and press **Save Information** when finished.

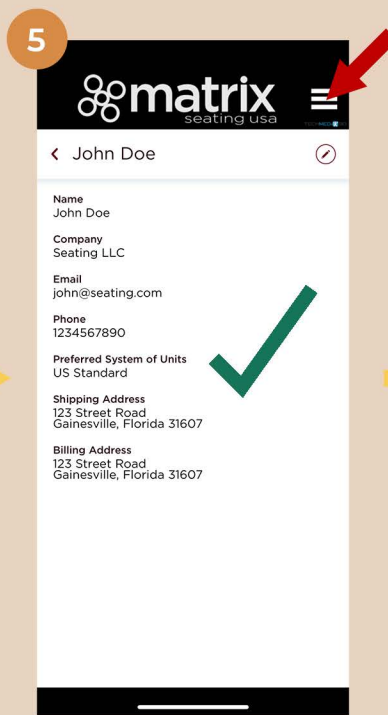


Verify the information is correct on the next page.

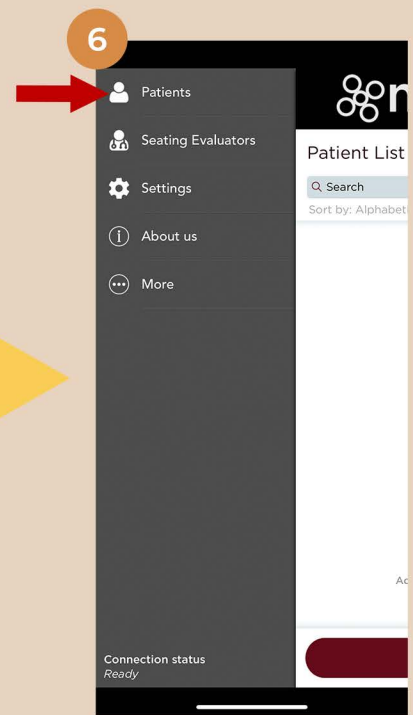


To add a new patient return to the **Patients List** by pressing the **Menu** button in the top right and pressing the **Patients** tab.

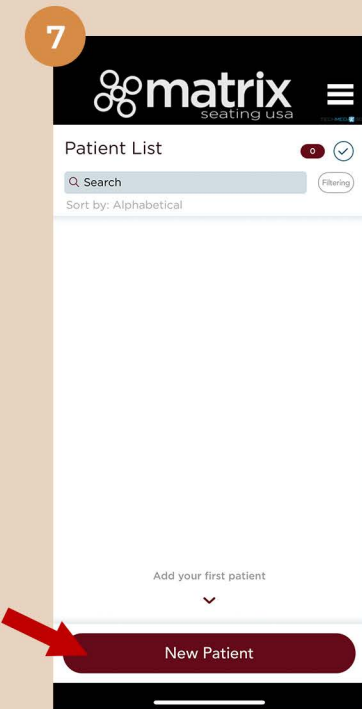

Press the **New Patient** button.

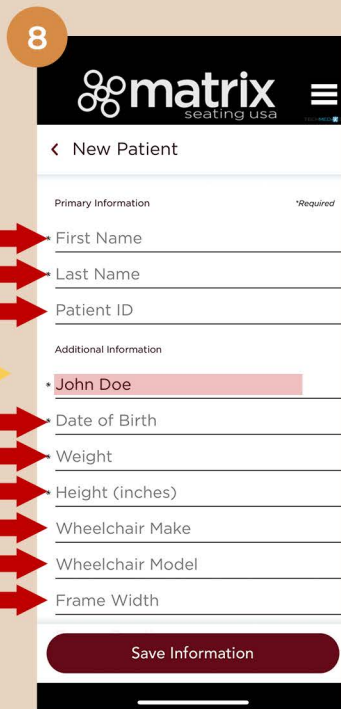


Fill out the required information and press **Save Information** when finished. **Ensure your name is displayed below patient ID.**

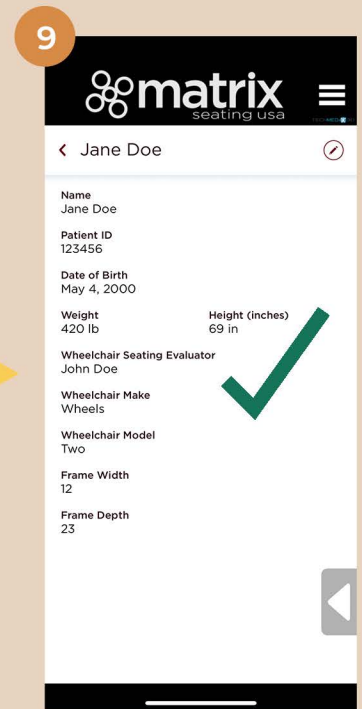


Verify the information is correct on the next page.

# Shape Capture

Keen3D for iPhone utilizes the front facing TrueDepth cameras. To scan using the iPhone start by facing the screen away from you, toward the object you are trying to capture. Once the scan is initiated it only takes a few seconds to capture the model. You may have to move the phone slightly to capture overhangs or fill in holes.

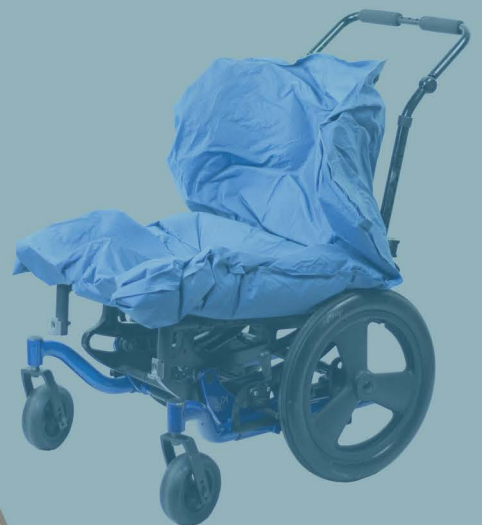




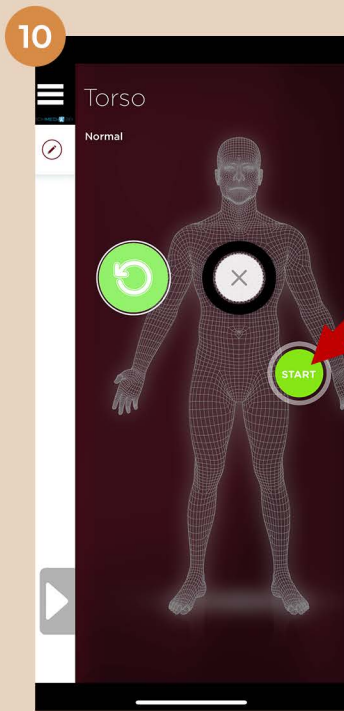


To begin a scan press the **Start** button.



Before proceeding, press the white arrow in the bottom left of the screen.

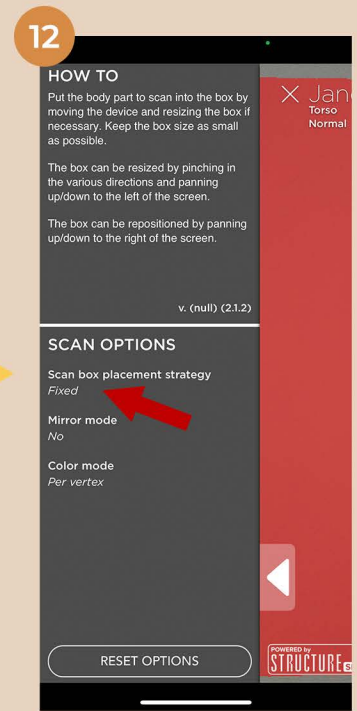


Under scan options press the **Scan box placement strategy**.

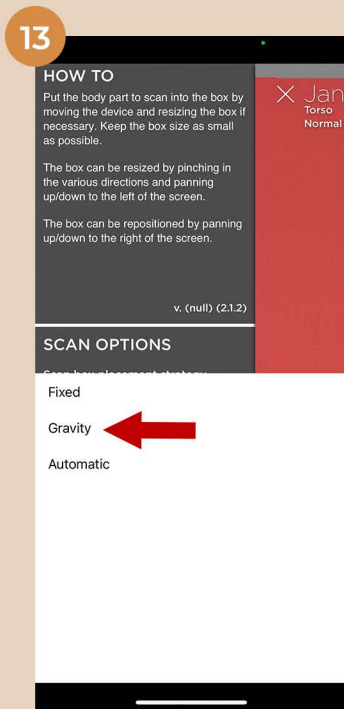

Change the strategy from Fixed to **Gravity**. This will allow you to scan larger objects with the iPhone app.



Once you are back on the scanner screen your shape capture should be mostly highlighted in red. If not, try moving closer.

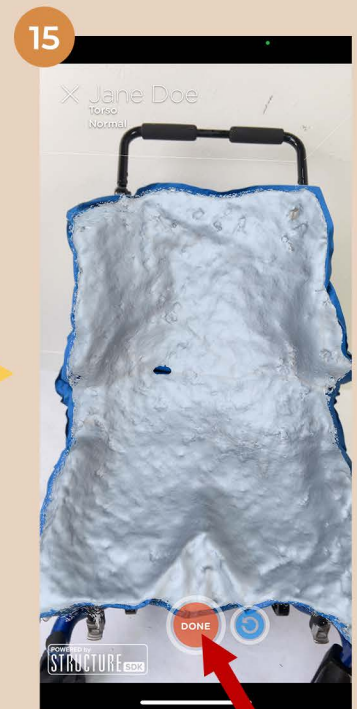

Press **Start** when you are ready to scan. Captured data will appear white. When the shape is fully covered in white, press **Done**.




Check to make sure your scan does not have any large holes or missing features. If you need to recapture, press the **Blue Arrow**. Press **Done** to proceed.




You can press the **clipboard** button in the lower tray to add scan notes.




You can press the **iPad** button in the lower tray to draw on the scan and add text. Press the **floppy disc** to save your notes.

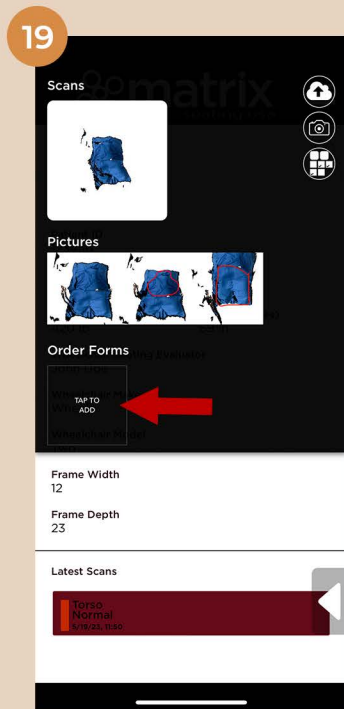

After pressing **Done** this page will appear. Press the box labeled **TAP TO ADD** to fill out an order form.

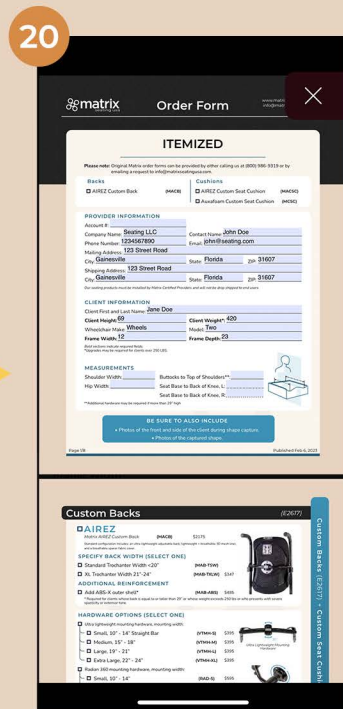

Fill in the order form and include as much information as possible. This will help us create the best system possible for your client!

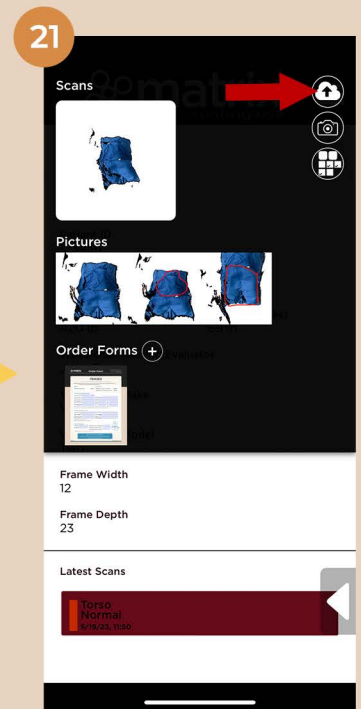

Ensure the information is correct and press the **Cloud** button in the top right corner. **This sends us the scan!**

All of the information you filled out is saved in the app. If you are at a location with poor service you can return to the app later and upload the scan under better conditions.

Patients in the **Patients List** will have a colored block before their name.

**Green** = Scan has been submitted **AND** received

**Yellow** = Scan has been submitted but **NOT** received

**Red** = scan has not been submitted

