


SHAPE CAPTURE MARK-UP ●●●●

Use these standardized mark-ups directly on the shape capture bag to improve accuracy and consistency. Clear markings streamline the transition from capture to molding, reduce field adjustments, and support faster delivery of a precise final product.

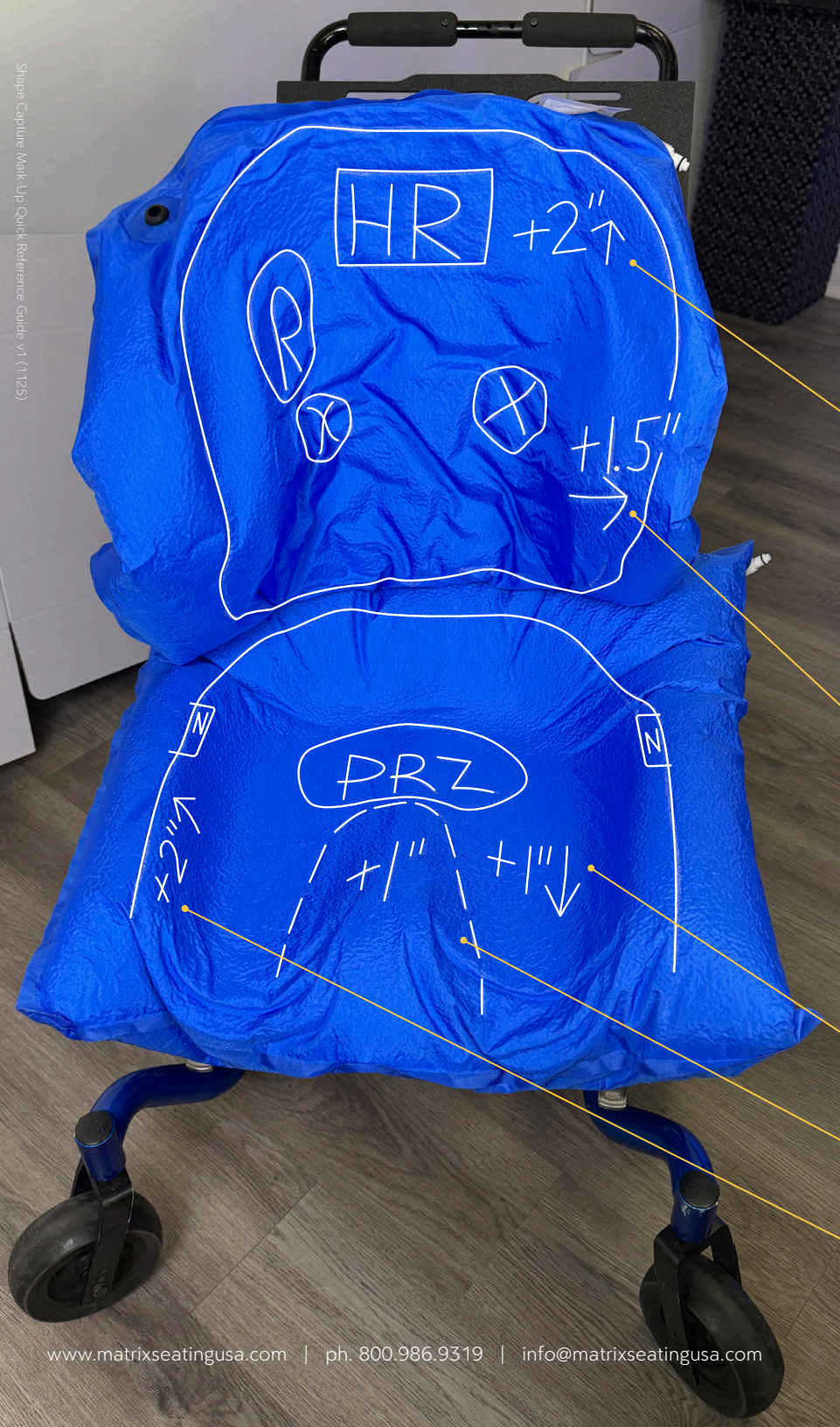

HR

Headrest placement*

*Marked on front; mounting bracket placement will be transferred to rear surface

+#”↑

Add back support height

+2”↑ Add 2 inches of back height

(R)

Add Relief Area

At a Gibbus, Rib Hump, or for cushion offloading

(X)

Add T-nuts

For Chest Strap or Harness

+#”← or +#”→

Increase width of lateral

+1.5”→ Extend lateral by 1.5 inches

N

Add Notches (seat belt, back canes, etc)

Indicate appropriate location and shape

PRZ

Add Pressure Relief Zone (soft spot)

Outline with appropriate shape

+#”↓

Add depth for leg length discrepancy

+1”↓ Add 1 inch of depth at leg trough

+#”

Abductor/Pommel location and size

+1” Add 1 inch of height/cushion thickness

+#”↑

Add height at lateral thigh area

+2”↑ Add 2 inches of lateral thigh height